
BESPOKE
STAIRLIFTS

SYNERGY

STRAIGHT STAIRLIFT USER GUIDE

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May 2025

1.0 Contents

1.1	Introduction	4
1.2	About Your Stairlift	6
1.3	Care and Safety	10
1.4	Maintenance	14
2.0	Your Synergy Stairlift	16
2.1	General Operation	20
2.2	Swivelling the Seat	29
2.3	General Information	32
2.4	Remote Controls	34
3.0	Hinged Rail Operation	40
4.0	Troubleshooting	50
5.0	Engineer Information	56
6.0	Warranty information	58
7.0	Service Record	62
8.0	Wiring Diagrams	66

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1.1 Introduction

Before using your stairlift, please read through these instructions carefully.

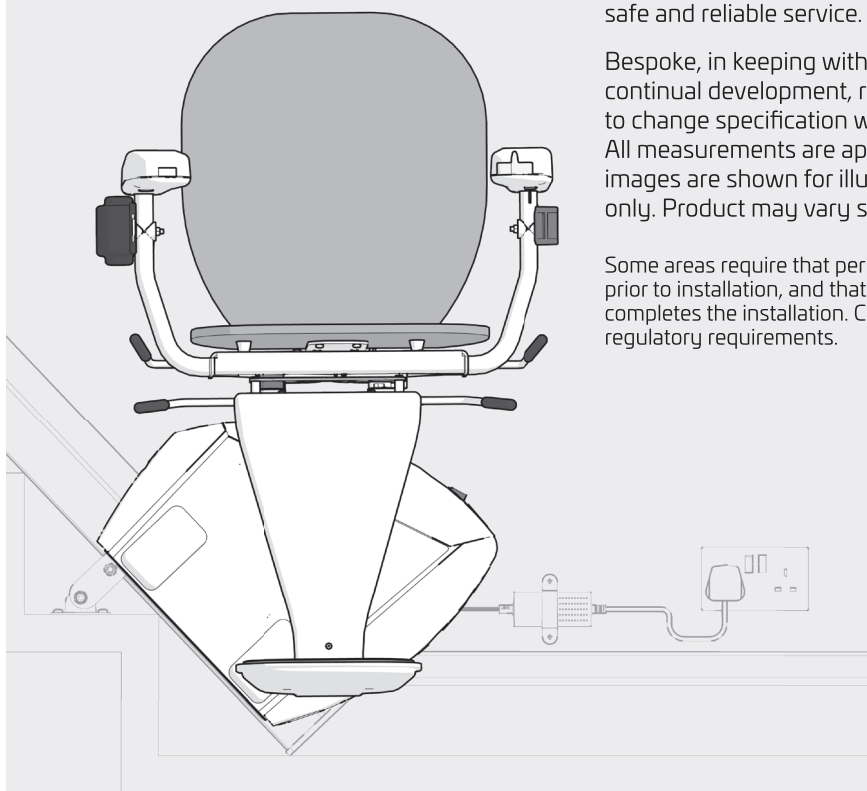
Thank you for choosing Bespoke Stairlifts. Your stairlift will provide many years of reliable service and allow you to enjoy the full use of your home.

Your stairlift is covered by a manufacturer's warranty that covers the cost of replacement parts. Please refer to your warranty certificate for the terms of your warranty.

Problems arising from the misuse or unauthorised modification of the equipment are not covered by the manufacturer's warranty. Your stairlift is designed and manufactured to the very highest standards. Annual maintenance is required to ensure safe and reliable service.

Bespoke, in keeping with its policy of continual development, reserves the right to change specification without prior notice. All measurements are approximate. All images are shown for illustrative purposes only. Product may vary slightly.

Some areas require that permits are obtained prior to installation, and that a licensed engineer completes the installation. Check your local regulatory requirements.

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1.2 About your Stairlift

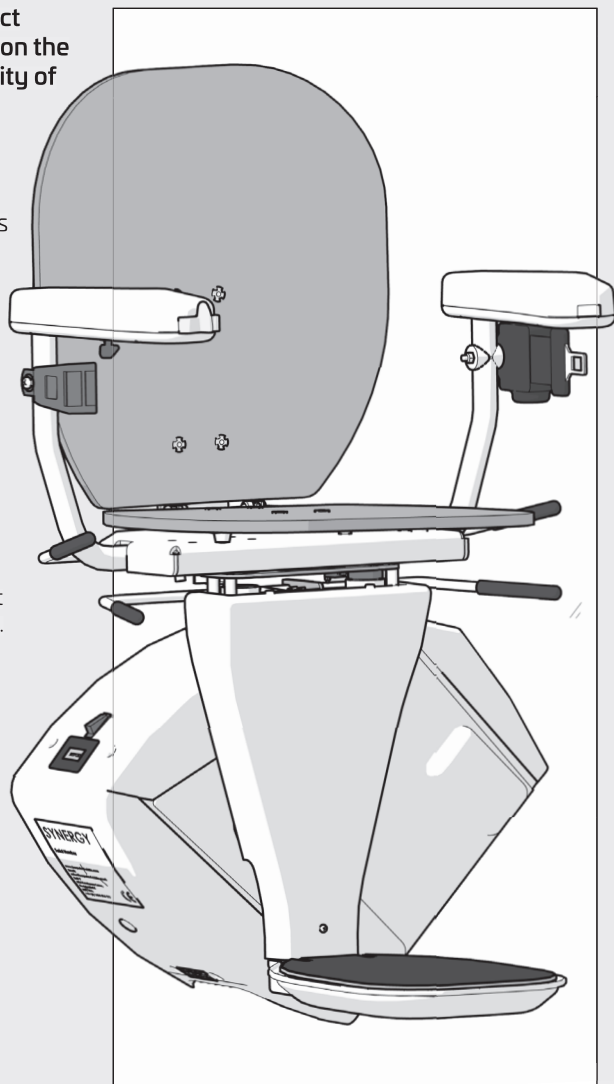
Your stairlift is a sophisticated product designed to carry one person sitting on the chair up to a maximum weight capacity of 125kg.

Your stairlift has been constructed utilising state-of-the-art technology, in accordance with EN81:40 standards and recognised safety regulations.

Incorrect use of the stairlift may result in risk to the user or others or possible damage to your stairlift or other goods. Please follow user instructions in accordance with this user guide. Keep this user guide close and within easy reach.

Should a problem occur which is not mentioned in this handbook please contact your stairlift supplier. The service department will always be pleased to assist and help you. It is important that you have the following information available when you are making a call:

- Your manual.
- Your full contact details (including Customer No.)
- Your lift serial number or purchase reference.

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1.2 About your Stairlift

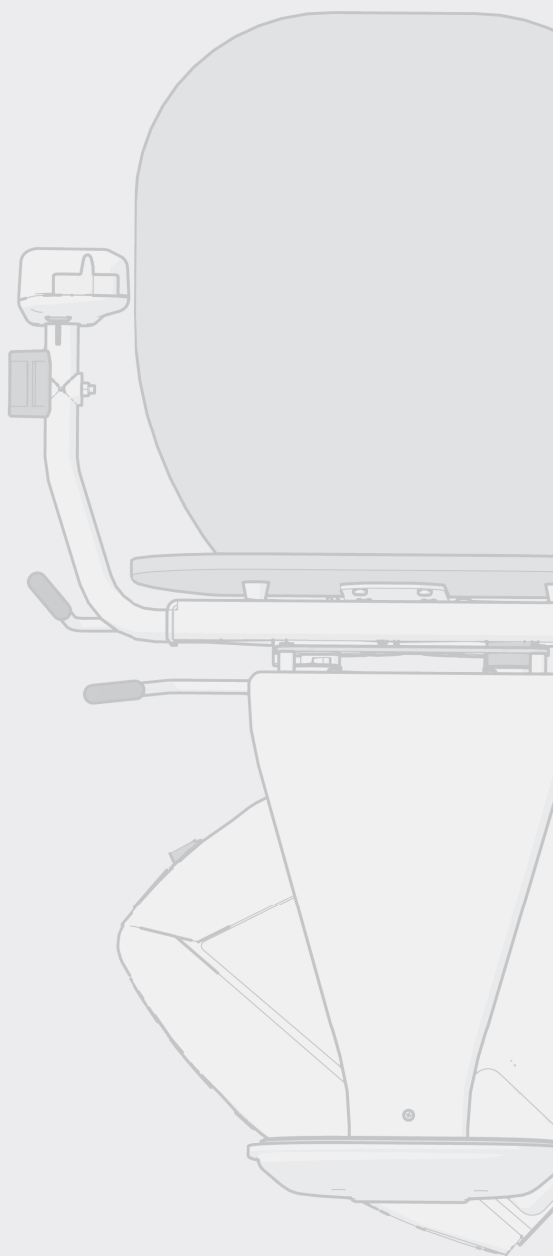
As the user of this stairlift, you are fully responsible for the observance of locally prevailing safety regulations and guidelines. The manufacturer warranty period is stated on your warranty certificate.

Any material or components breaking down during that period will be replaced free of charge. (excluding batteries)

Exclusion from warranties: Damage caused by normal wear and tear, overloading or misuse, replacements of batteries in the remote controls.

The carrying out of any alterations to the product is not permitted and will affect your warranty and also the lift may no longer be compliant with our standards. The manufacturer is not liable for damage resulting from such alterations.

The manufacturer will accept no liability for damage or injury caused by inadequate observance of the safety regulations and safety instructions in this manual; or by negligence during installation, use, maintenance and repair of this stairlift. Installation and servicing may only be carried by responsible installers who are certified and authorised by the manufacturer, during which the necessary regulations must be observed.

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1.3 Care and Safety

Ensure you have been given a thorough demonstration by the installation engineer.

When using the stairlift always observe locally prevailing safety regulations regarding risk of electric shocks and physical injury.

The stairlift should only be used by one person at the time.

Do not use the stairlift for other than intended use.

Make sure that other household members or visitors know the stairlift is in use and are aware of any potential trip hazards.

Do not use the stairlift for evacuation during a fire.

Be aware that small objects can fall onto the stairlift rail and may slide into the mechanism. If you suspect something is trapped, contact your Bespoke representative before using the stairlift again.

Disconnecting the power supply will deplete the stairlift's battery charge, rendering it inoperable.

Ensure a safe seating position. Sit as far back as possible back into your chair, use the armrests and place the feet squarely on the footrest.
Do not stand on your Stairlift.

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1.3 Care and Safety

Ensure that there are no obstructions where the stairlift needs to be installed.
The entire track must be free of obstacles. Check with your installer if you are unsure.

Take environmental influences into account such as damp or extreme sunlight. The entire track must be visible to avoid tripping.

Keep unauthorised persons and domestic pets away from the stairlift.
The lift is not designed to carry animals or goods.

Keep children away from your stairlift unless under supervision of an adult.
Please remember that your lift is not a toy.

Remove the key from the keyswitch if the lift is not in use.

Keep limbs and clothing away from moving parts. Make sure that loose clothing cannot be trapped or caught between the chair or rail.

Ensure the seatbelt is fastened securely at all times during use.

If you feel that your stairlift is defective, contact your local dealer or installer.

Only use qualified engineers to carry out any repairs on the lift.

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1.4 Maintenance

General Maintenance

- Check and clean the stairlift regularly
- Avoid excessive use of water
- Clean the seat, arm and footrest with only a damp cloth

Once a year, it is advised to call your Bespoke Stairlifts representative or authorised dealer to perform the annual inspection, that includes all maintenance and safety inspections and repairs where necessary.

Regular maintenance is essential for keeping your stairlift in proper operating condition. We strongly recommend that you only use authorised engineers to perform all required maintenance, service and repair work.

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
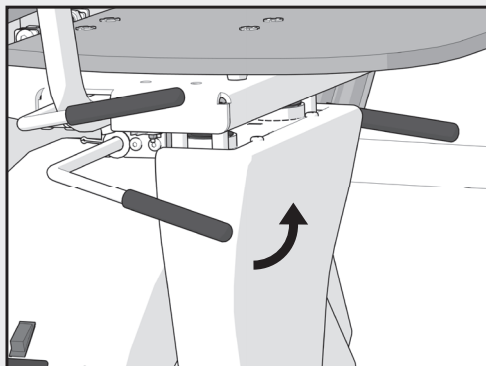
2.1 General Operation

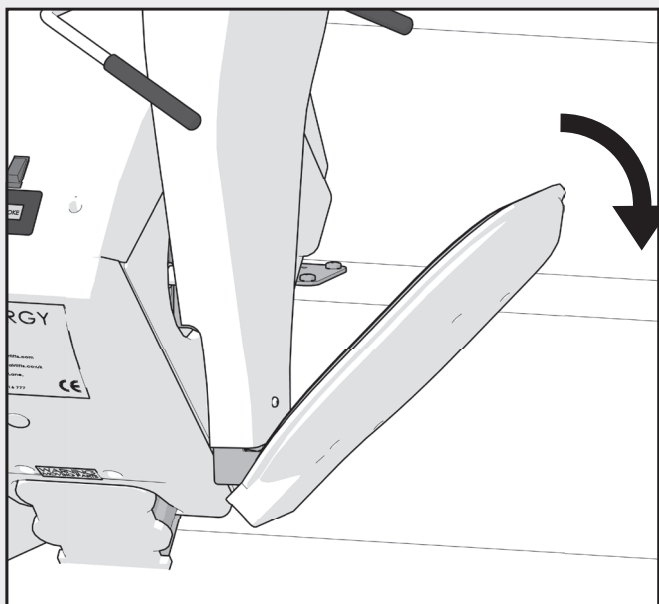Gently lower the footrest using the footrest lever.

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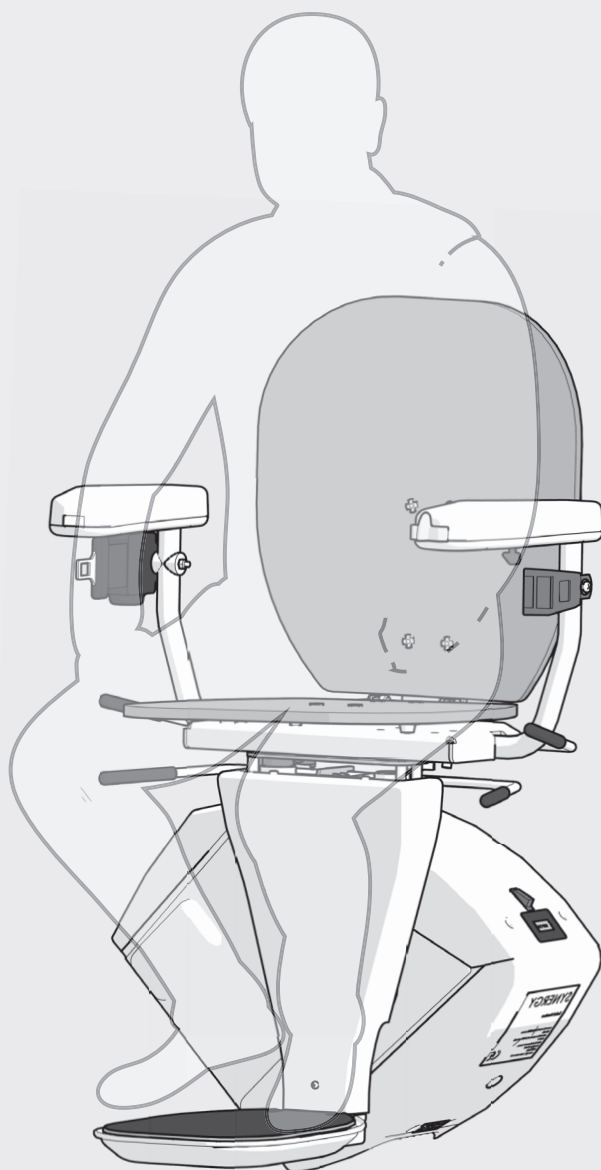
2.1 General Operation

**Take a seat in
the stairlift and
make yourself
comfortable.**

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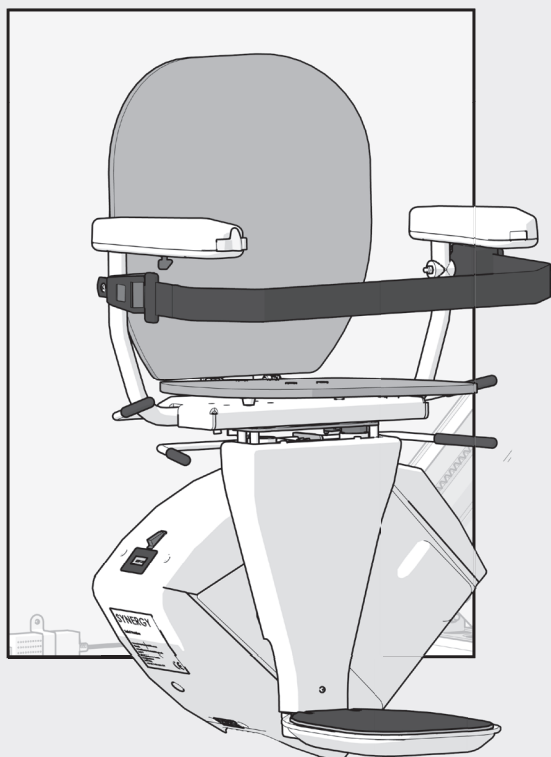
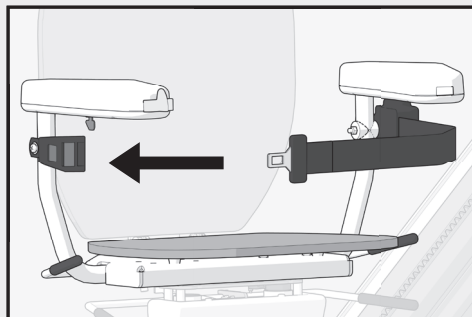
2.1 General Operation

**Please ensure
the seatbelt
is securely
fastened.**

**Your stairlift is
now ready to
move up or down
the stairs**

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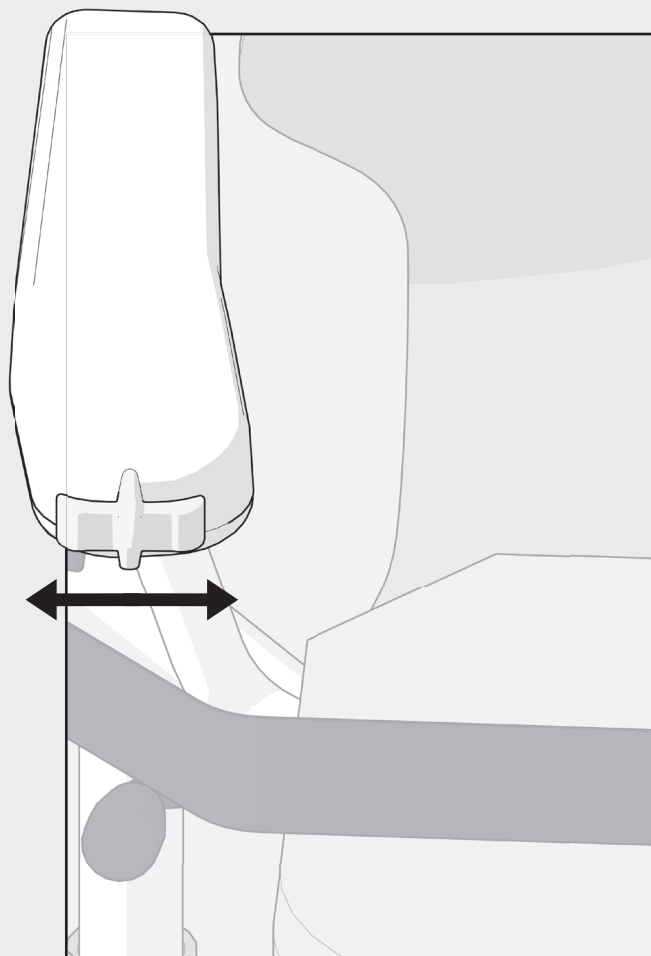
2.1 General Operation

Press the directional toggle in the direction you wish to travel.

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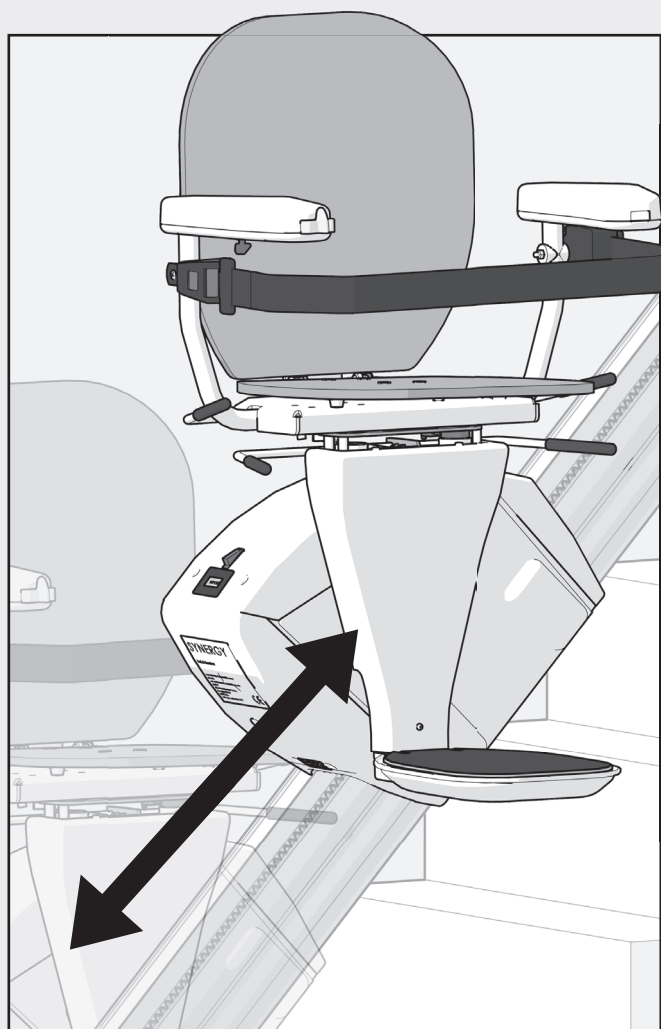
2.1 General Operation

Continue to hold the toggle switch in the direction of travel until you reach the end of your journey.

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2.1 General Operation

Please ensure your stairlift has fully completed its journey to ensure the lift is charging correctly.

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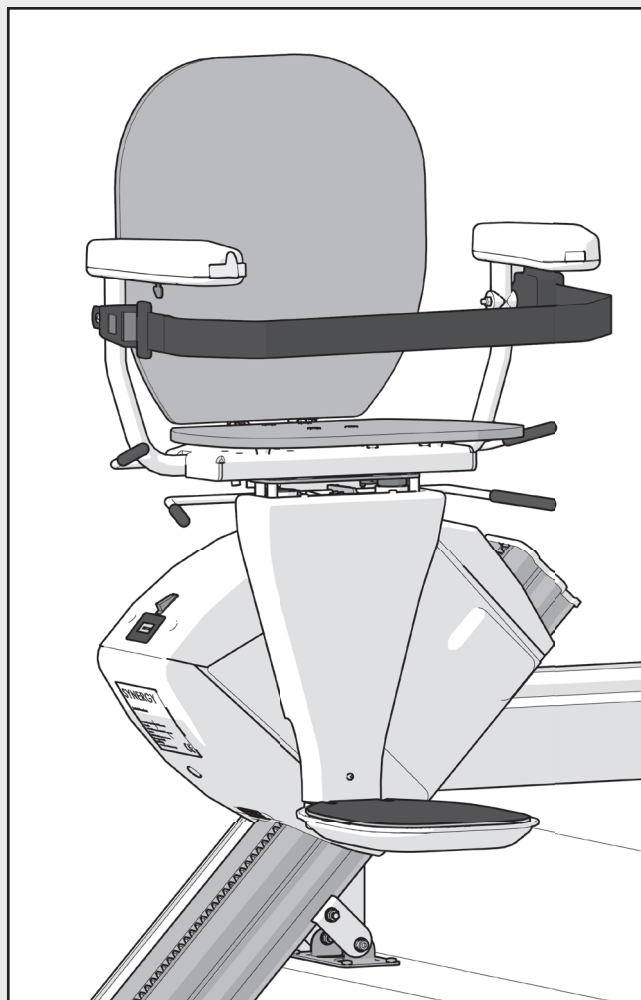
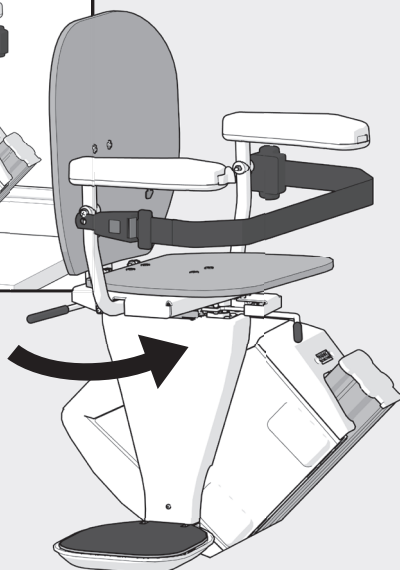
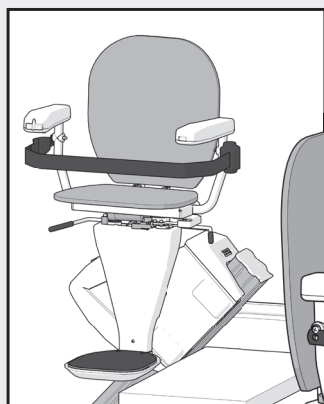
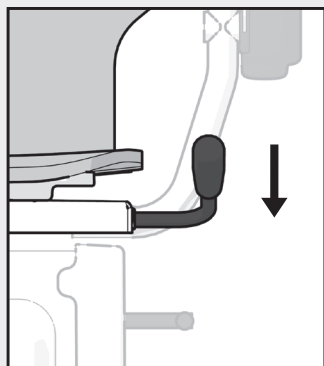
2.2 Swivelling the seat

Manual Swivel Operation

Depress the lever under the seat and swivel the chair safely away from the stairs.

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
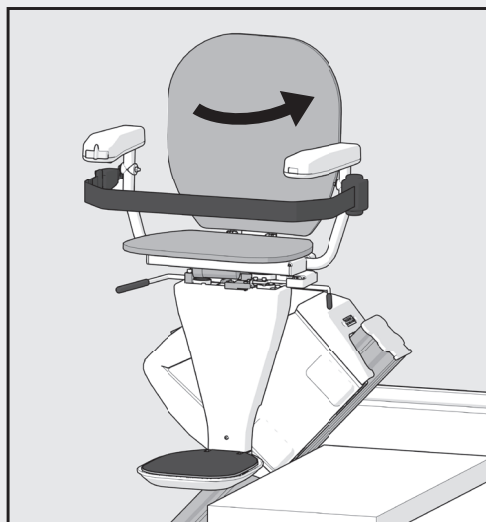
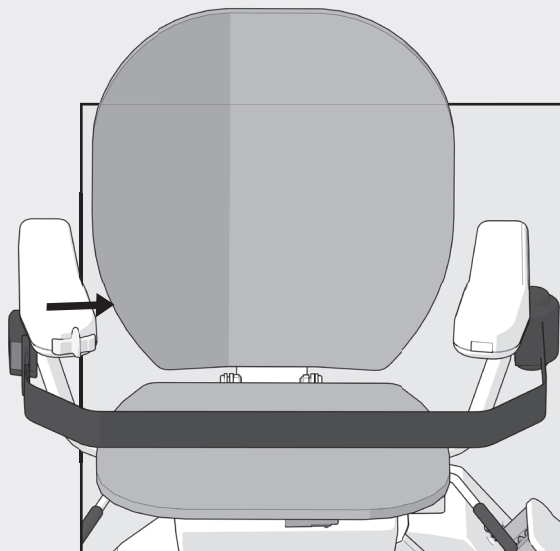
2.2 Swivelling the seat

Automatic Swivel Operation

Hold the toggle switch in the direction of travel until the chair has completely swivelled.

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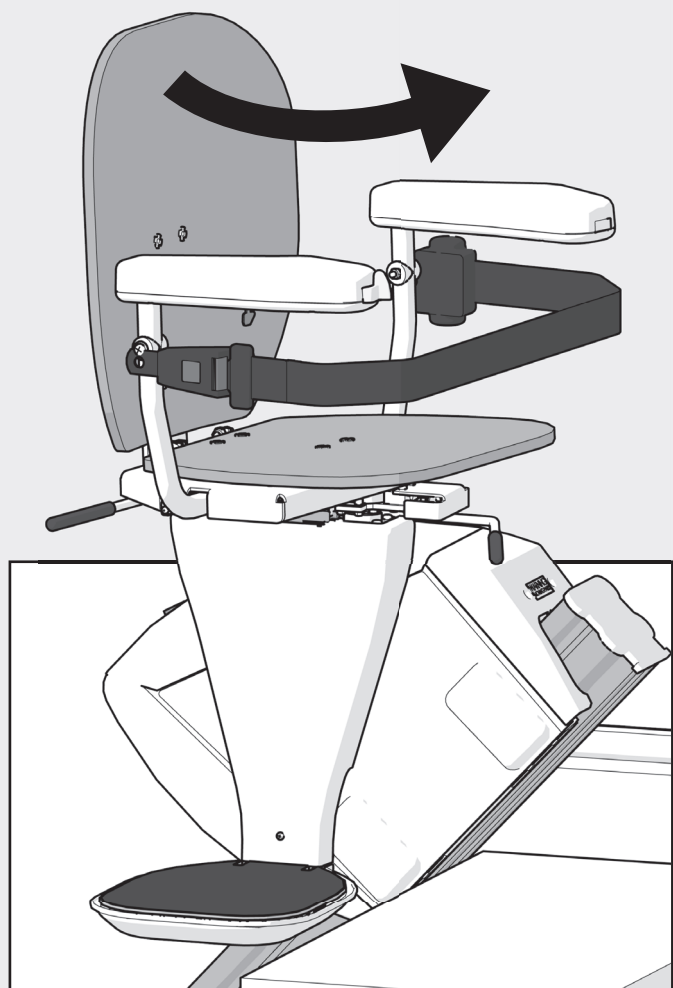2.3 General Information

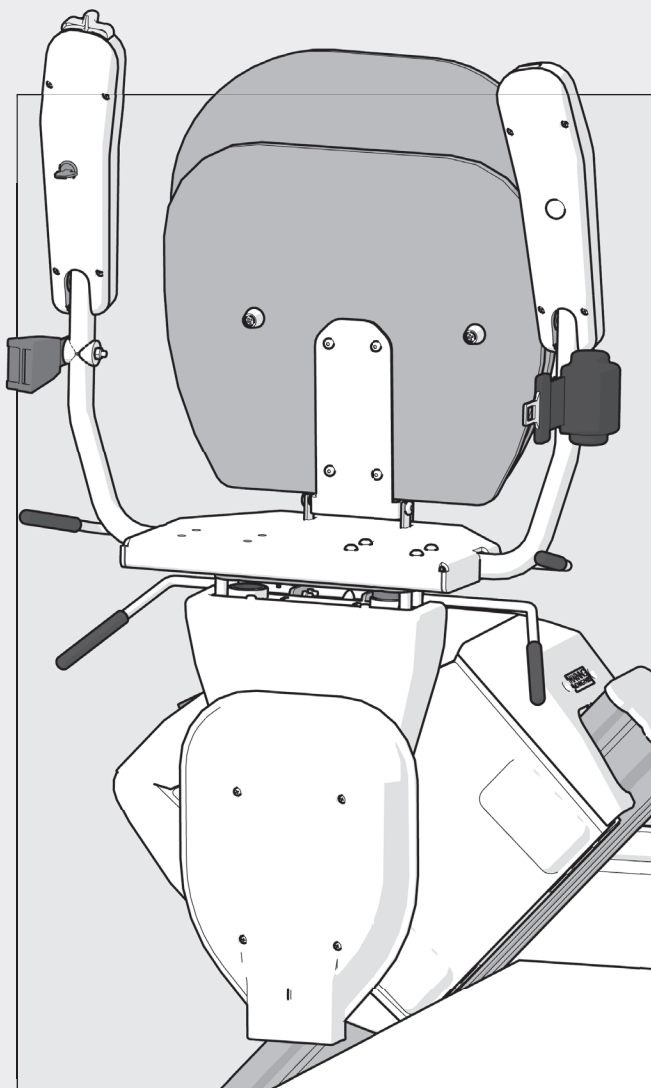
When not in use:

Your stairlift should be parked securely on a charge point.

Return swivel seat to the original position.

Fold up seat, arms and footrest.

If the stairlift is not charging correctly an audible beep will sound.

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2.4 Remote Controls

1 UP Button
Moves the stairlift up

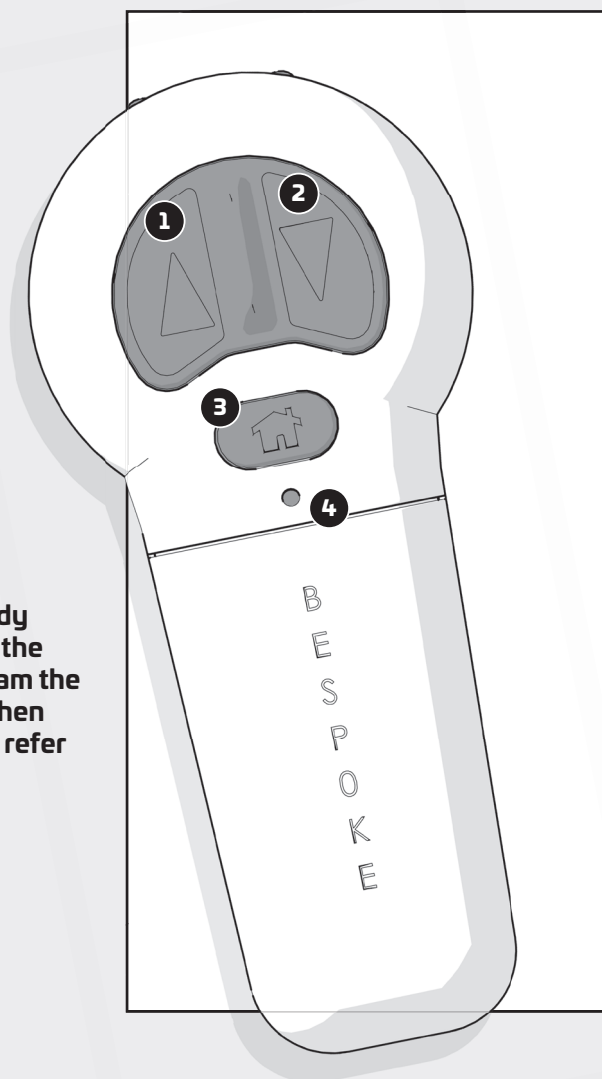
2 DOWN Button
Moves the stairlift down

3 HOME Button
Sends the stairlift to an intermediate charge point.*

4 RESET Button

Your remote control comes ready to use with your stairlift, but in the event that you need to reprogram the remote control (for example, when replacing the batteries), please refer to the following instructions.

*If required. For example, if a hinged rail is being used.

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2.5 Programming the Remote

Your remote control takes 2 x AA batteries.

To insert the batteries, remove the screw holding the battery cover in place, and remove the battery cover.

Insert new batteries, and replace the battery cover.

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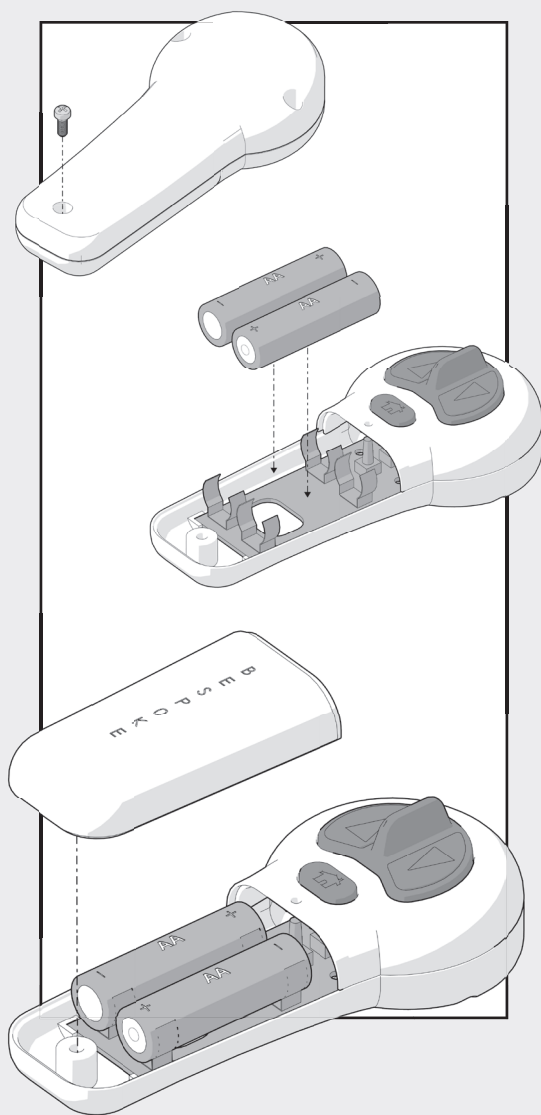
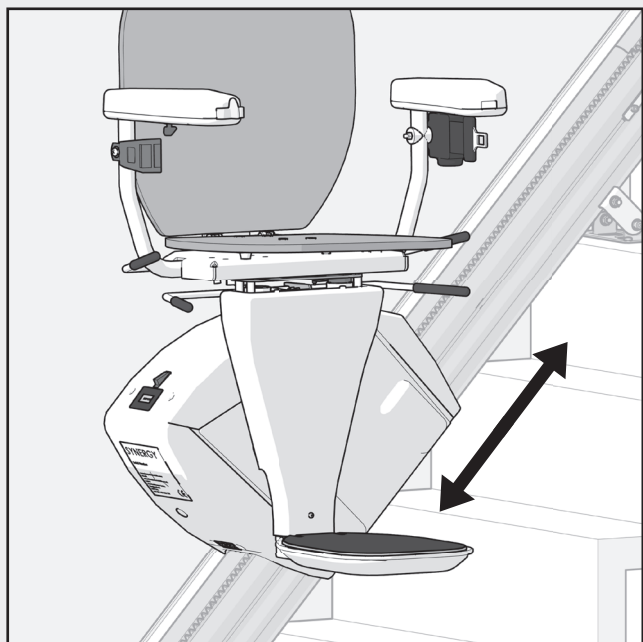
2.5 Programming the Remote

Move the stairlift off a charging point using the directional control on the stairlift armrest.

Turn the stairlift off at the main power switch, and then back on.

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
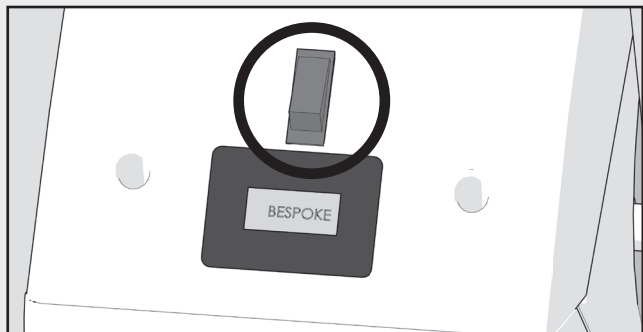
2.5 Programming the Remote

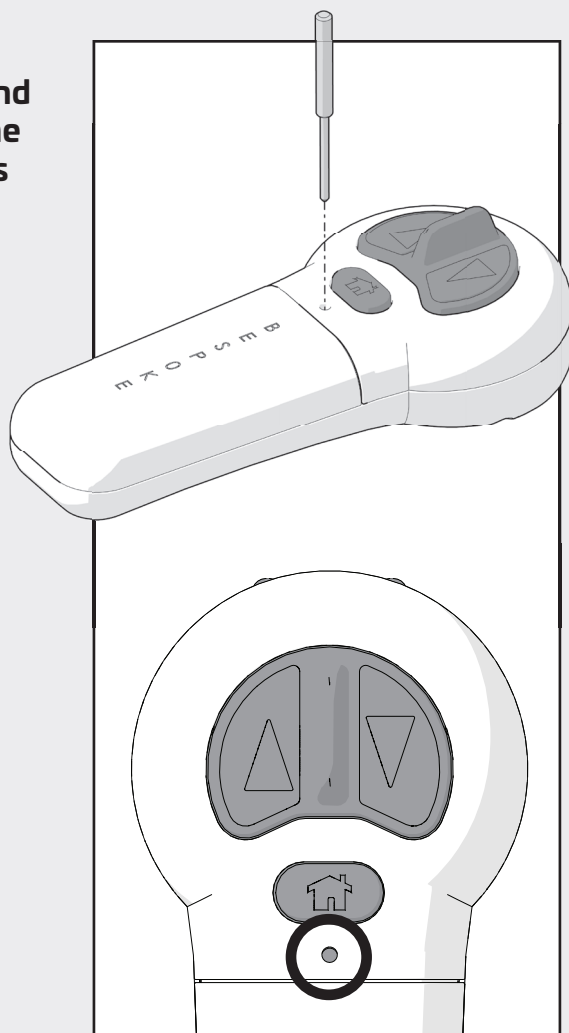Using a small blunt object such as a paper clip, press and hold the RESET button on the remote control for 3 seconds

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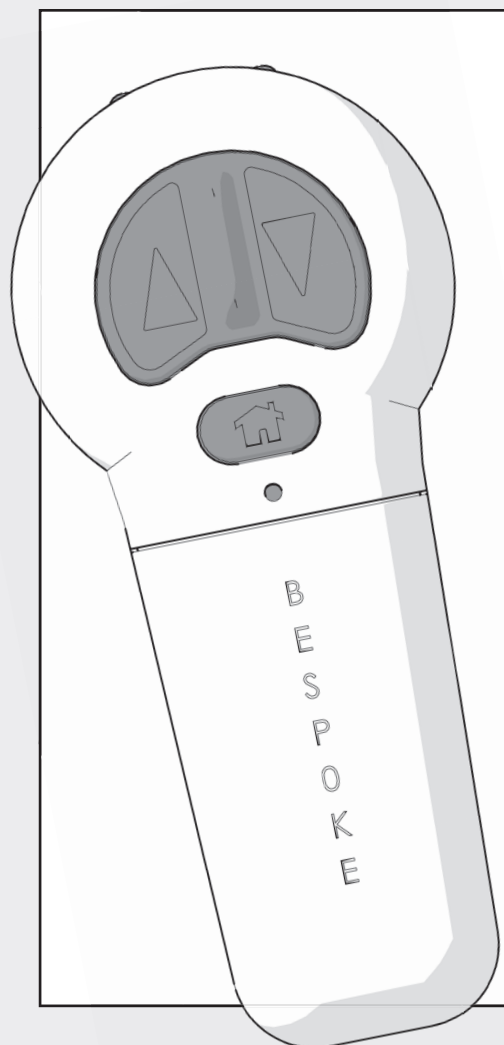
2.5 Programming the Remote

Press any directional button on the remote control handset.

Your remote control is now paired to your stairlift.

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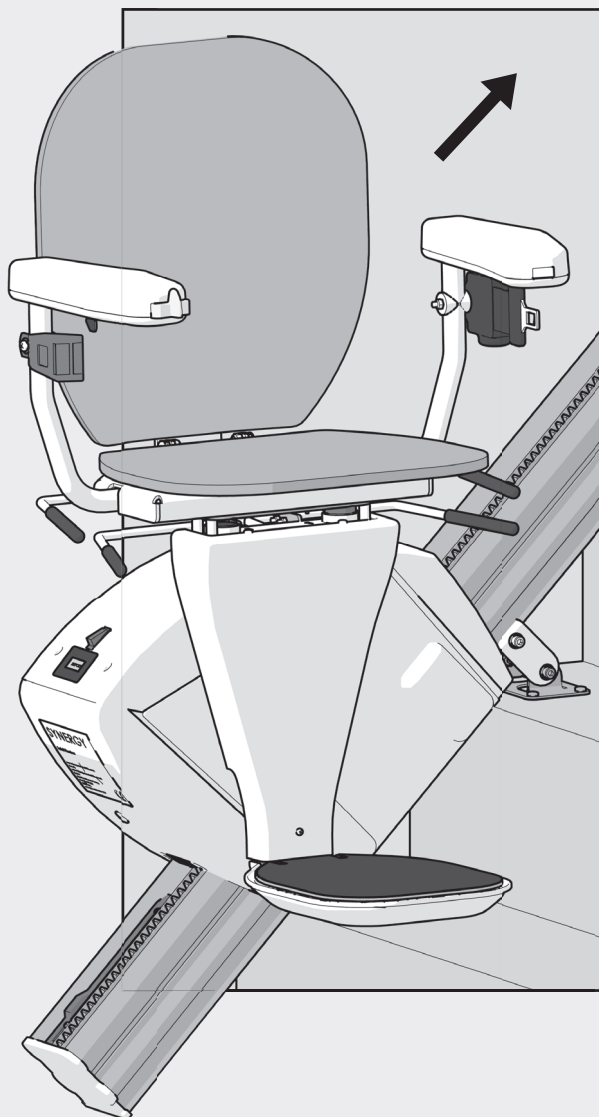
3.0 Hinged Rail Operation

**Move the stairlift
'UP' by pressing
the toggle switch
in the desired
direction.**

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
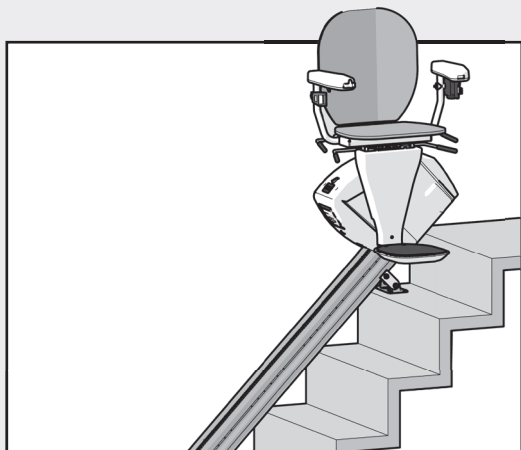
3.0 Hinged Rail Operation

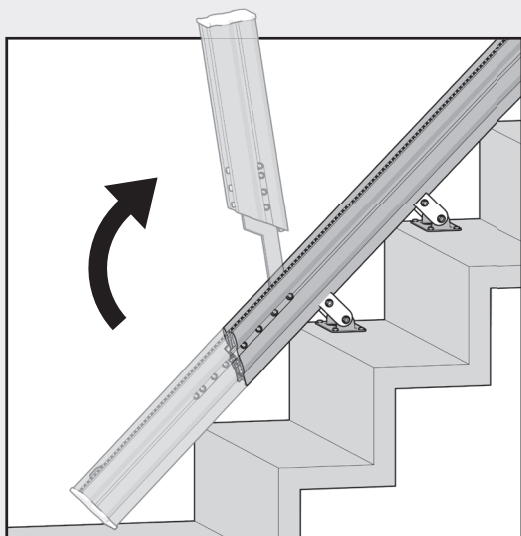Once the stairlift reaches the top of the staircase, keep the control pressed to begin the hinge raising procedure.

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3.0 Hinged Rail Operation

Continue pressing the 'UP' control until the hinged section of the rail is in the fully upright position.

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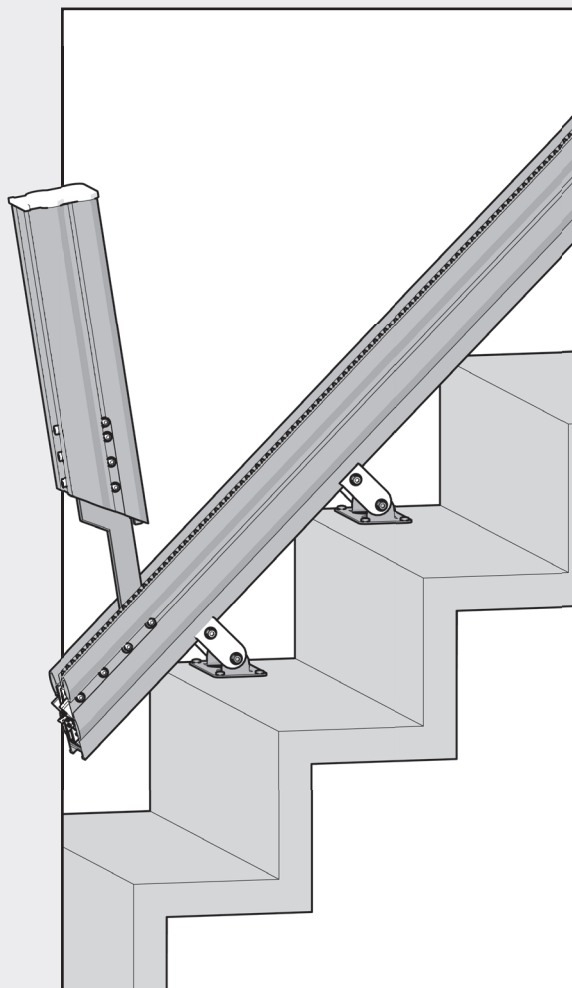
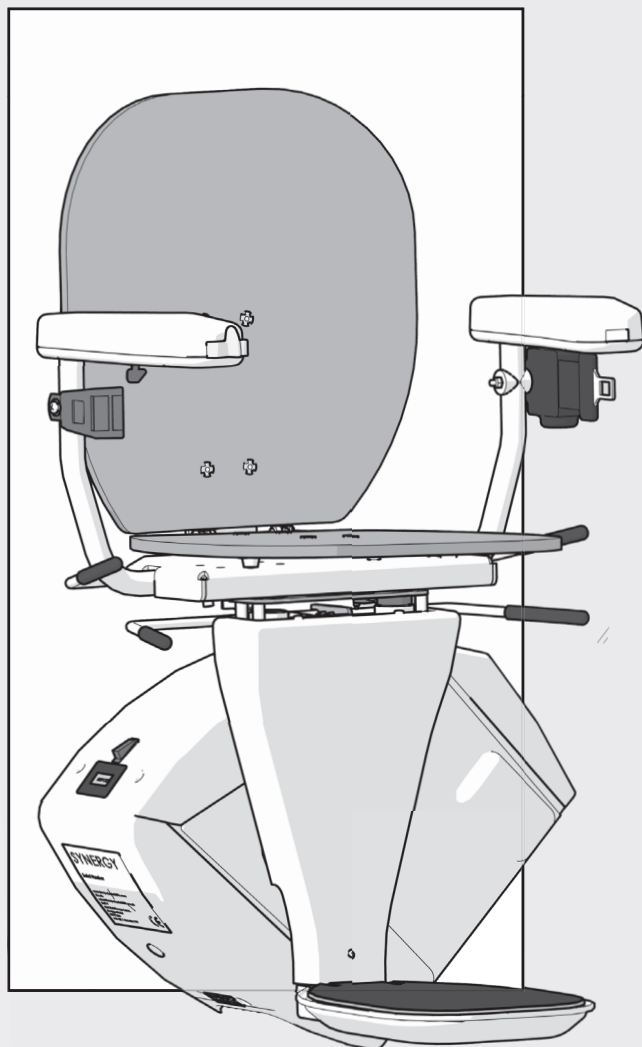
3.0 Hinged Rail Operation

**Once complete,
it is now safe to
leave the stairlift
as normal.**

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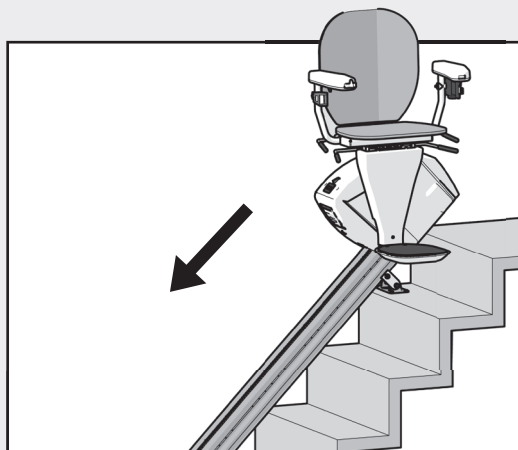
3.0 Hinged Rail Operation

To return to the lower floor, press the control in the 'DOWN' direction.

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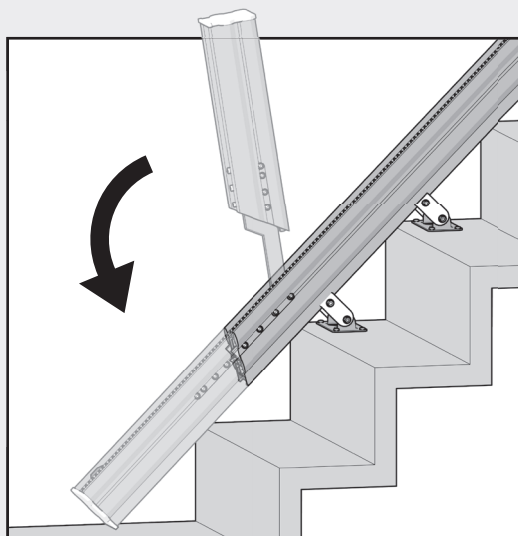
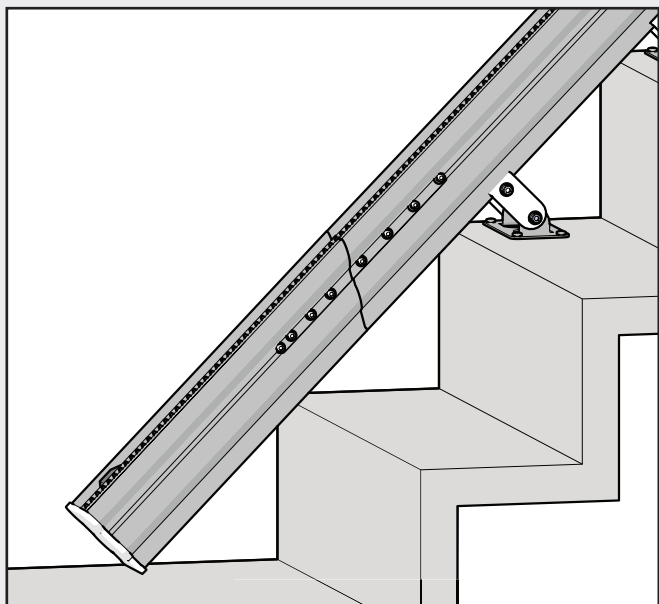
3.0 Hinged Rail Operation

Keep the directional control pressed. Once the hinge has returned to the fully closed position, the stairlift will descend automatically.

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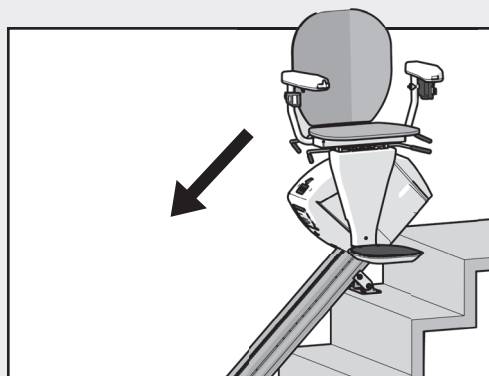
3.0 Hinged Rail Operation

Once the stairlift reaches the bottom parking point, the stairlift will gently stop.

It is now safe to get off the stairlift as normal.

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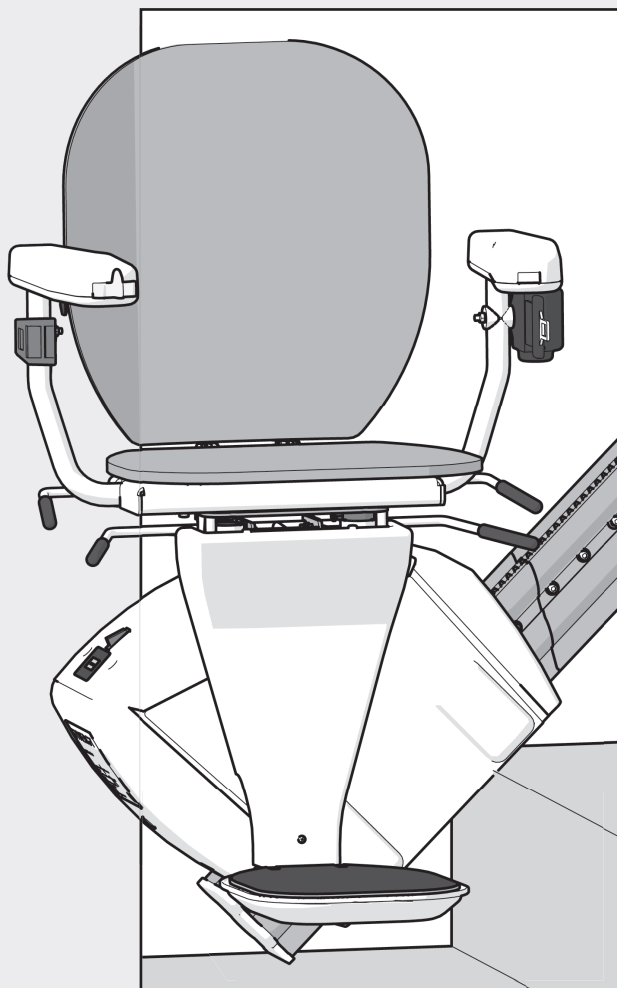
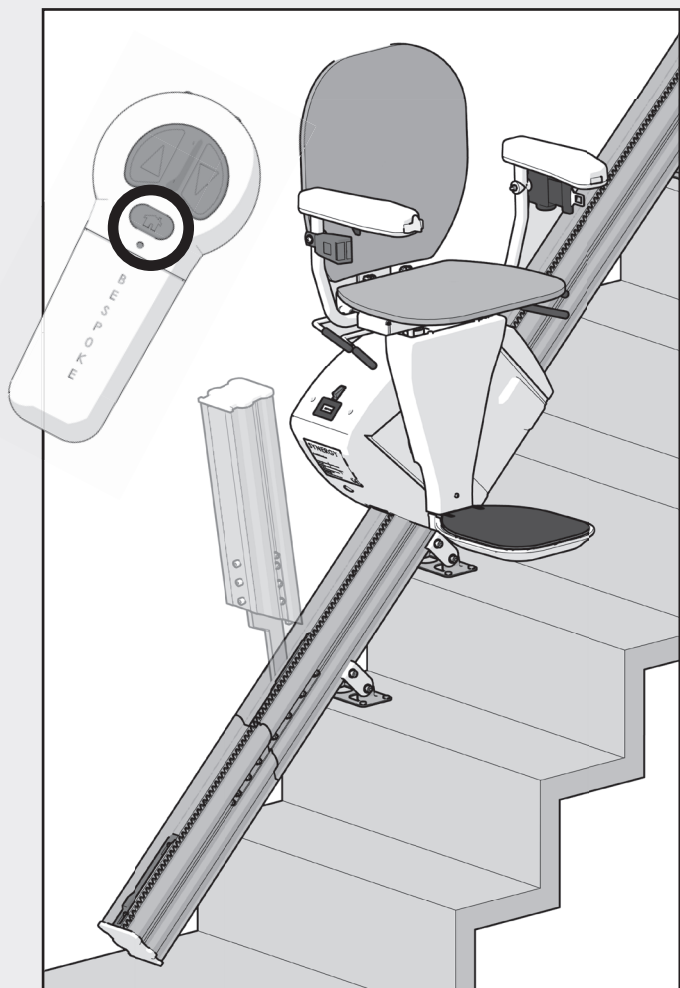
3.0 Hinged Rail Operation

To park the stairlift on an intermediate parking position (if installed), press the 'HOME' button on the remote control. Once parked, the hinge will raise.

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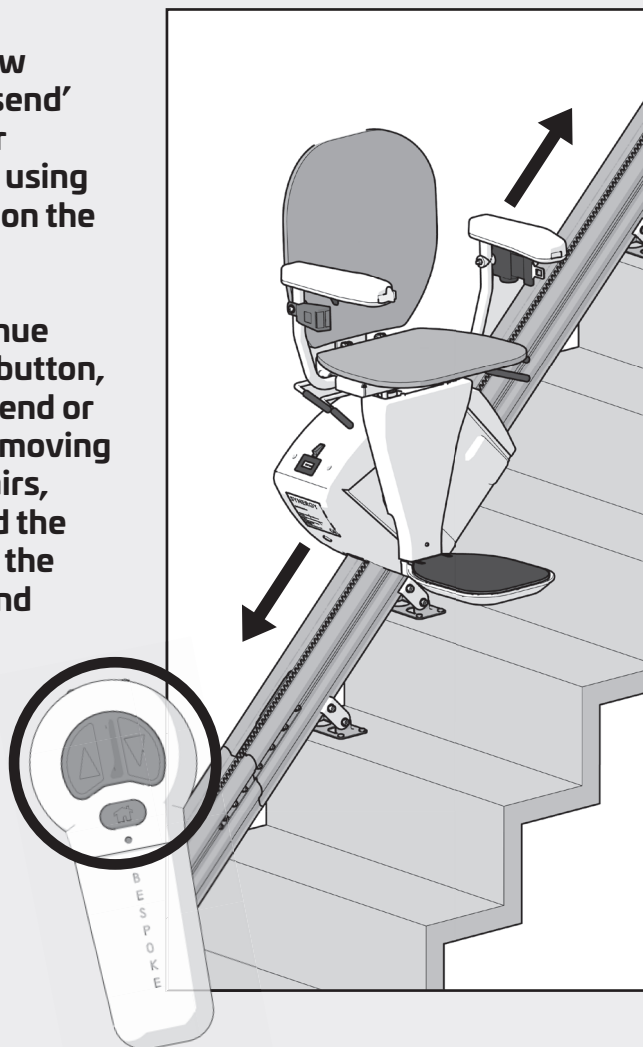
3.0 Hinged Rail Operation

When you wish to use the stairlift, you can now subsequently 'call' or 'send' the stairlift to the top or bottom of the staircase using the directional buttons on the remote control.

Simply press and continue to hold the up or down button, and the stairlift will ascend or descend as required. If moving to the bottom of the stairs, the hinge will lower and the stairlift will continue to the bottom parking point and gently stop.

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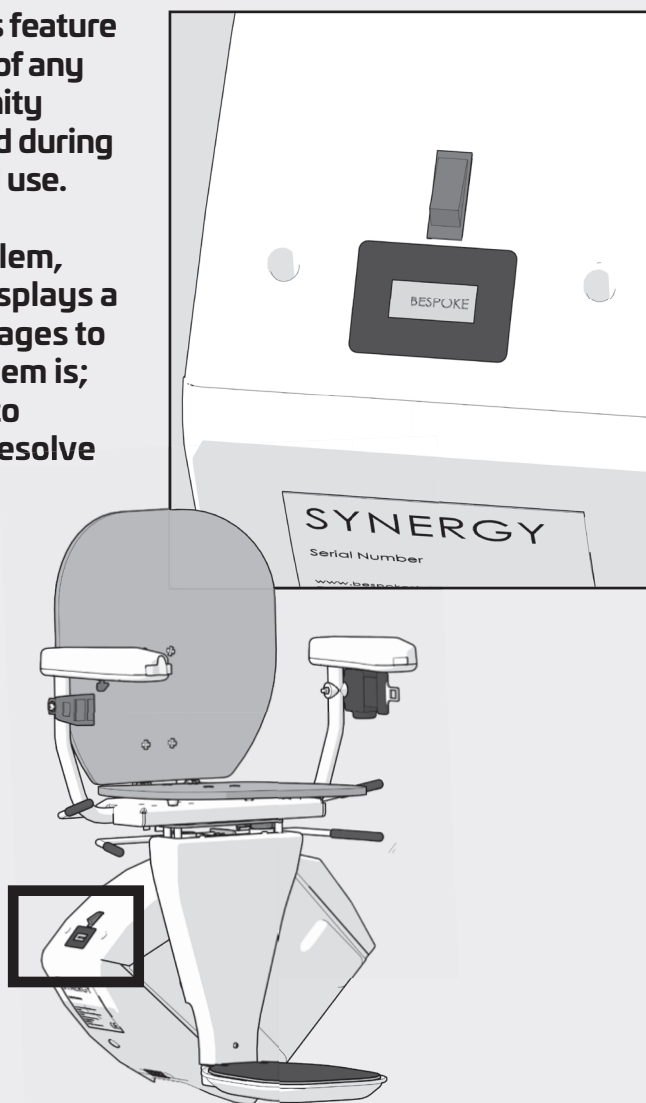
4.0 Troubleshooting

The Display Diagnostics feature can inform an engineer of any problems that your Infinity Stairlift has encountered during installation or in general use.

Depending on the problem, the Synergy Stairlift displays a selection of error messages to indicate what the problem is; enabling the engineer to pinpoint the fault and resolve it quickly.

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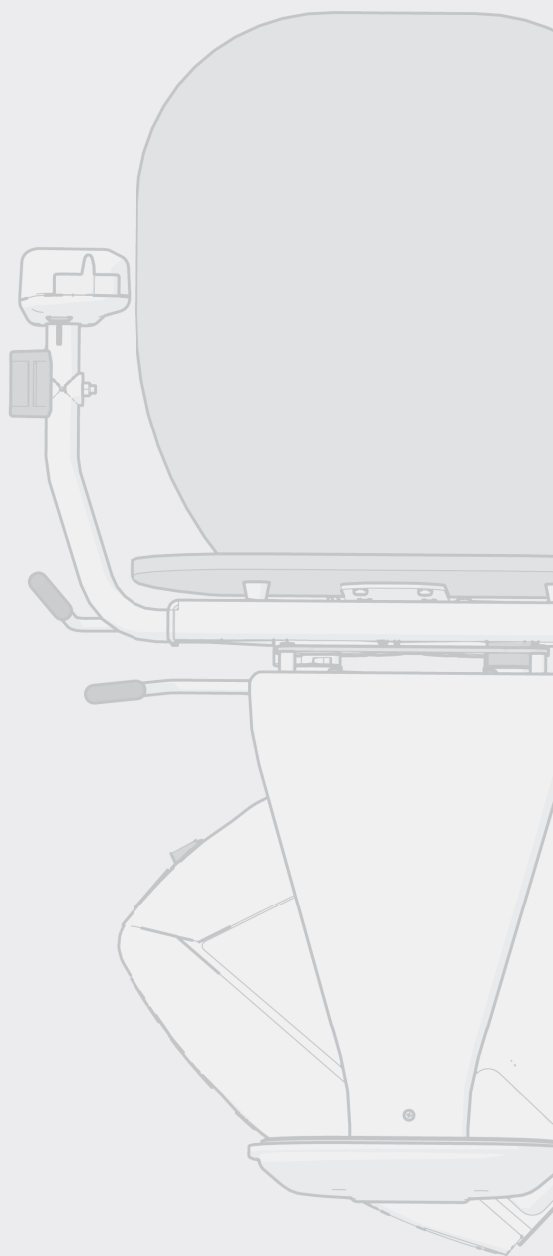
OFF CHARGE

Lift not charging

- Check lift is parked on a charge point
- Check the charger has not been turned off.

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FOOTREST LEFT/RIGHT

Footrest safety switch activated

- Check for obstructions and ensure covers are able to move freely

FOOTREST
LEFT

FOOTREST
RIGHT

4.0 Troubleshooting

SEAT LEFT/RIGHT

Main chassis safety switch activated

- Check for obstructions and ensure covers are able to move freely

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4.0 Troubleshooting

SEAT SWIVEL

Seat swivel safety switch activated

- Contact Installer/Dealer
(Will also show when powered swivel is in motion)

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4.0 Troubleshooting

ADDITIONAL FAULTS NOT SHOWN ON DISPLAY

Lift will only work from remote controls

- Check both arms are horizontal
(arm interlock is present in downside arm)
- Check key switch is turned on
(wiring for interlock is in series with the key switch)

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
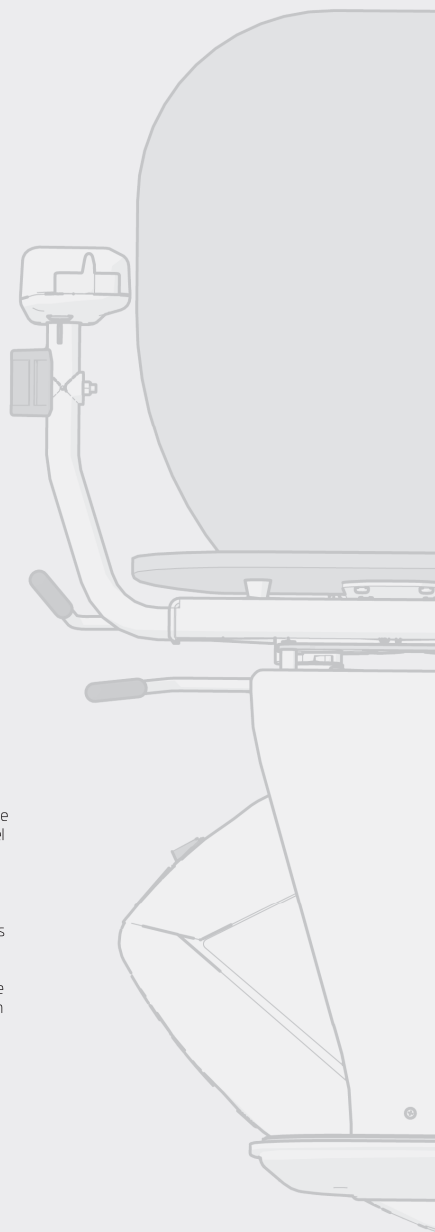
5.0 Engineer Information

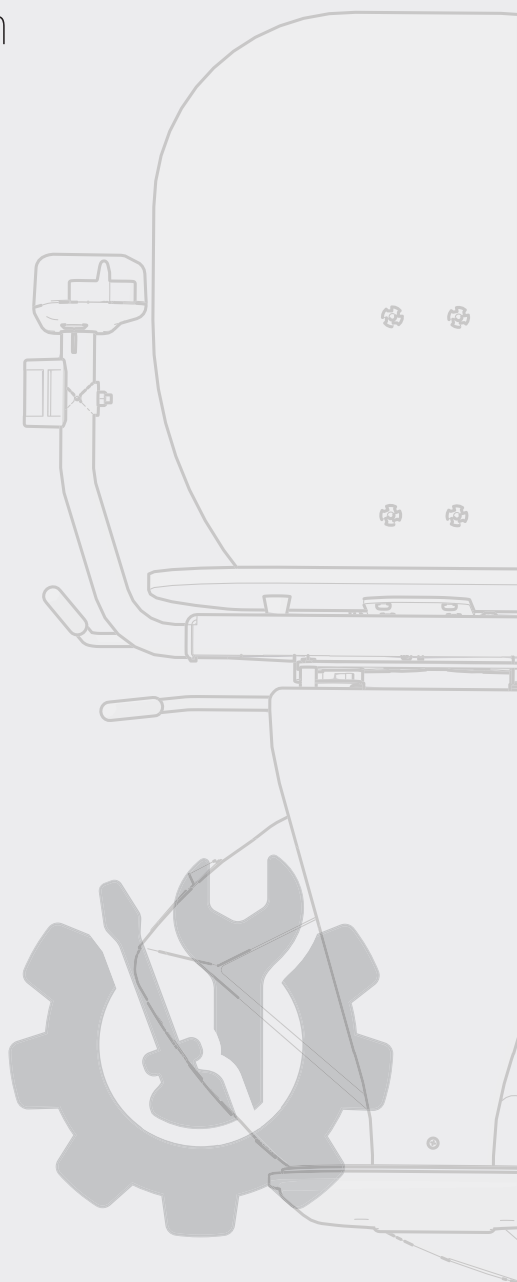- Check the lift for any damage.
- Check that all Parts and packages are on site.
- Ensure that you have your installation guide book on site.
- Inspect all parts are in full working order.
- Only use original lift parts (this could potentially affect any warranty).
- Ensure all safety devices are fitted correctly, these may only be removed if maintenance or repair is needed and only be serviced by a qualified Engineer.
- The lift should not be used or left if the safety devices are not functional or incomplete.
- The safety devices must be checked regularly to see that everything is in working order, if not these should be repaired immediately.
- Use the device in accordance with intended use.
- Ensure PPE is in place when necessary, i.e. shoes, safety glasses, hard hat, gloves during the assembly and servicing of the lift.
- Remember to hold spare parts and the correct tools in your vehicle.

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6.0 Warranty Information

Thank you for choosing Bespoke Stairlifts. Your stairlift will provide you with many years of reliable service and allow you to enjoy the full use of your home.

Your stairlift is covered by a manufacturer's warranty that covers the cost of replacement parts. Please refer to your warranty certificate for details of your warranty. Ensure that only an authorised and qualified Bespoke Stairlifts trained engineer installs and services the stairlift.

Under no circumstance should anyone other than an engineer trained and authorised to install, adjust, service or modify any mechanical or electrical device on this product. Failure to follow this warning may result in safety system compromises which could result in serious injury or death. Bespoke Stairlifts accepts no liability for property damage, warranty claims or personal injury, including death, in this circumstance.

After installation, reliable operation and continual safe operation requires regular service and inspection. As the owner, you are responsible for ensuring that regular service and inspections occur in a timely manner.

As well as ensuring your stairlift receives its recommended annual service, taking out an extended aftercare package will give you priority attention throughout the year. It means that, in the unlikely event of a breakdown, one of our dedicated team will be on hand to deal with any queries.

Call our direct Aftercare team on 01274 009757 for further details, or contact your authorised dealer.

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WARRANTY CERTIFICATE

SYNERGY STRAIGHT STAIRLIFT

Congratulations, you have purchased a Bespoke Synergy Stairlift.

Once your purchase is complete, and your stairlift has been installed, you will have a full manufacturer's warranty for 12 months, or for the period specified below. Please retain this certificate along with your dated receipt to ensure your warranty is valid.

WARRANTY COMMENCEMENT DATE:

VALID FOR A PERIOD OF MONTHS UNTIL:

Customer No:

--	--	--	--	--	--	--	--	--	--

Serial No:

--	--	--	--	--	--	--	--	--	--

Bespoke Stairlifts
Unit 9, Old Fieldhouse Industrial Estate,
Old Fieldhouse Lane, Huddersfield,
West Yorkshire, HD2 1FA, England

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7.0 Service Record

Regular Stairlift Maintenance

To maintain the ongoing advantages of your Synergy Stairlift, it's essential to ensure that it continues to provide reliable support. Just like any machinery with moving components, regular servicing and maintenance are necessary. We suggest scheduling an annual visit from one of our skilled service engineers to guarantee that your stairlift performs as efficiently as it did on the day of installation.

Installed By:		On Behalf of:		Date:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	
Date:	Service Details:		Engineer:	Next Service Due:	

Bespoke Stairlifts requires all their stairlifts to be serviced and maintained in line with their manufacturer's recommendations and a thorough examination must be undertaken at the appropriate intervals for the particular equipment type. Number of service inspections required per annum for stairlift products are 1 per 12 months. These services should be carried out in line with the written maintenance program ensuring compliance to international standards by qualified engineers. Copies of all test reports and service logs must be retained for a minimum of 5 years by the head office of the company servicing the product.

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7.0 Service Record

Maintenance Advisory Checklist - Synergy Straight Stairlift

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‘Y’ indicated recommended frequency of checks, maintenance and/or replacements.

PLEASE NOTE: Substitution of any wire or current-carrying device for the proper fuse or circuit breaker in a lift control circuit shall not be permitted. The substitution of a device or component in its entirety with a new unit that is basically the same as the original for the purpose of ensuring performance in accordance with applicable Standard requirements. Bespoke Stairlifts advises replacement parts should be provided only by the manufacturer.

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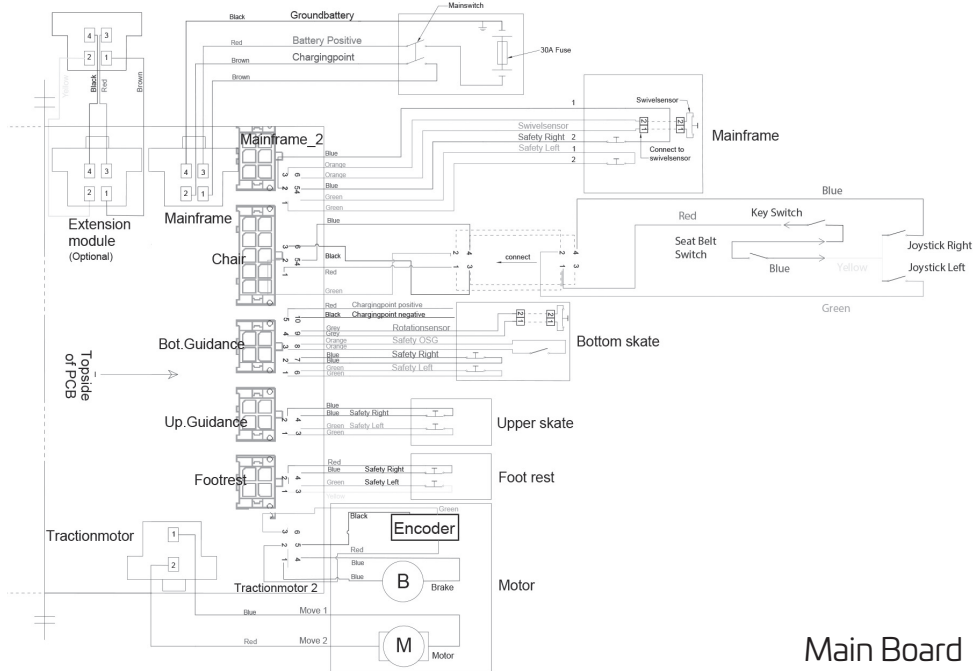
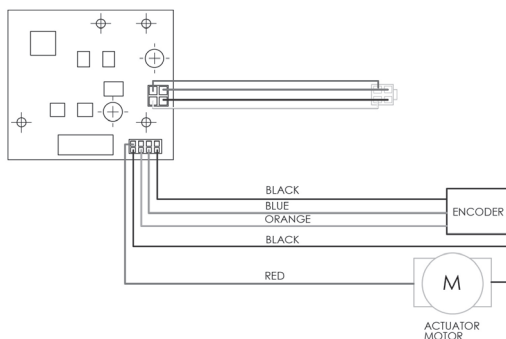
CERTIFICATE OF CONFORMITY

Bespoke Stairlifts of

Unit 9, Old Fieldhouse Industrial Estate,
Old Fieldhouse Lane, Huddersfield,
West Yorkshire, HD2 1FA, England

Declare that under our sole responsibility that the products:

SYNERGY STRAIGHT STAIRLIFT

have been manufactured in conformity with the
following standards and specifications:

EN 81-40:2020

And the products to which this declaration relates
are in conformity with the following EU directives:

Electro Magnetic Compatibility Directive - 2014/30/EU

Low Voltage Directive - 2014/35/EU

Machinery Directive - 2006/46/EC

Signed:

on behalf of Bespoke Stairlifts Limited

Date: 1st June 2024

Technical documentation is available on request
from Bespoke Stairlifts at the address above.

BESPOKE
STAIRLIFTS

2410121/USM/SYN/0525/11

Bespoke Stairlifts Ltd.
Unit 9, Old Fieldhouse Industrial Estate,
Old Fieldhouse Lane, Huddersfield,
West Yorkshire, HD2 1FA, England

www.bespokestairlifts.co.uk

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STAIRLIFTS LTD.